BRAUN

Blood Pressure Monitor
BP4900
BP5900

ExactFit / ExactFit Plus

English 3
Español 12


Fig. 1

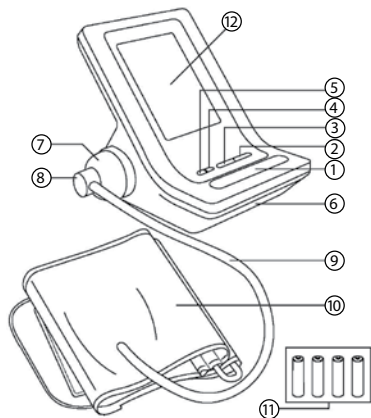Product Description


1. Start/Stop Button 
2. Memory Button 
3. Average Button 
4. Set Button 
5. Mode Button 
6. Battery Compartment Cover
7. Hose Port
8. Connector
9. Hose
10. Arm Cuff
11. 4x AA Alkaline Batteries (1.5V)
12. LCD Display

Internet: www.kaz.com/braun

This product is manufactured by Kaz Inc under a license to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany. ©2009 Kaz Incorporated.

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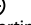
English

Braun upper arm blood pressure monitor has been developed for accurate and comfortable blood pressure measurements. The measuring accuracy of Braun's upper arm blood pressure monitor was tested at the time of manufacture and the measurement algorithm was proven by clinical research in accordance with AAMI/ANSI-SP10.

What you should know about blood pressure

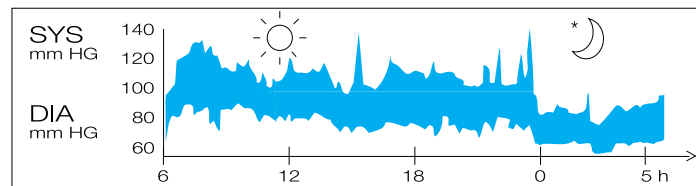Blood pressure constantly changes throughout the day. It rises sharply in the early morning and declines during the late morning. Blood pressure rises again in the afternoon and finally drops to a low level at night. Also, it may vary in a short period of time. Therefore, readings from successive measurements can fluctuate.

Blood pressure readings taken from a healthy 31-year-old male, measured at 5-minute intervals

Blood pressure measured in a doctor's office only provides a momentary value. Repeated measurements at home better reflect one's actual blood pressure values under everyday conditions.

Moreover, many people have a different blood pressure when they measure at home, because they tend to be more relaxed than when in the doctor's office. Regular blood pressure measurements taken at home can provide your doctor with valuable information on your normal blood pressure values under actual «everyday» conditions.

The World Health Organisation (WHO) has set up the following standard blood pressure values when measured at resting pulse.

Blood Pressure (mmHg)	Normal Values	Mild Hypertension	Severe Hypertension
SYS = systole (upper value)	up to 140	140-180	over 180
DIA = diastole (lower value)	up to 90	90-110	over 110

Intended Use

- This monitor is intended for use in measuring the blood pressure and pulse rate of adults and uses the oscillometric method.
- This product is intended for household use only.

Warnings & Precautions

- To ensure accurate measuring results, carefully read the complete use instructions.
- Keep product and batteries away from children.
- People suffering from cardiac arrhythmia, vascular constriction, arteriosclerosis in extremities, diabetes or users of cardiac pacemakers should consult their doctor before measuring their blood pressure themselves, since deviations in blood pressure values may occur in such cases.
- If you are under medical treatment or taking any medication, please consult your doctor first.
- The use of this blood pressure monitor is not intended as a substitute for consultation with your doctor.

Inserting Batteries

- Remove the battery compartment cover at the bottom of the unit and insert 4 AA LR6 ALKALINE batteries with correct polarity (see symbol in the battery compartment).
- Note: Always re-set Date and Time after replacing new batteries to make sure the measurement results are stored with correct date and time.
- Only discard empty batteries. They should not be disposed of in the household waste, but at appropriate collection sites or at your retailer.

Fig. 2

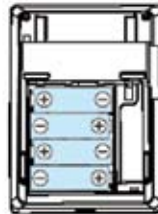
Fig. 3

Key Rules for Accurate Blood Pressure Measurement

- Always take readings at the same time of day, ideally in the morning and evening, under the same conditions.
- Please rest for at least 5~10 minutes before taking the measurement.
- Do not measure within 30 minutes after exercising, bathing, eating, smoking or consuming coffee, tea or alcohol.
- To allow your blood vessels to return to the condition prior to taking the measurement, please wait at least 3~5 minutes between measurements. You may need to adjust the wait time according to your personal physiological situation.
- Take off wrist watch and jewelry before fitting the cuff on the measuring arm.
- Do not use a cellular phone near the unit.
- While taking a measurement, sit down, relax, keep still, and do not move or speak.
- Wrap the cuff snugly around your arm. The cuff must be at heart level.
- Do not vibrate the unit during measurement, or the proper measurement will not be achieved.
- Perform measurement quietly in a relaxed position.
- Sit in a chair with your feet flat on the floor.

- Do not wrap the cuff over jacket or sweater sleeve or measurement cannot be done.
- Remove tight-fitting clothing from your left arm.
- Do not in any way twist the arm cuff.
- Do not inflate the monitor's cuff when it is not wrapped around the arm.
- Do not attempt to disassemble or change any parts of the monitor including the cuff.
- To avoid any possibility of accidental strangulation, do not drape the tube around your neck.
- Do not drop the product or put it through strong impact.
- The device is not supposed to be used if your arm has any wound or injury.

Applying the Arm Cuff

1. Slide the end of arm cuff furthest from the tube through the metal ring to a loop. The smooth cloth should be on the inside of the cuff.

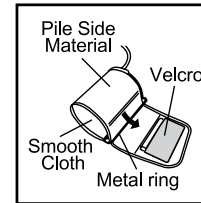
Fig. 4

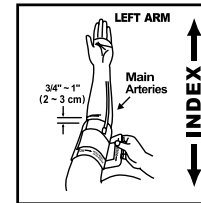
Fig. 5

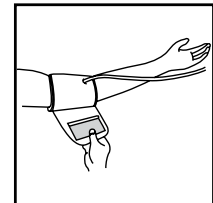
Fig. 6

2. If the cuff is located correctly, the Velcro will be on the outside of the cuff and the metal ring will not touch the skin. (Fig. 4)
3. Put your left arm through the cuff loop. The bottom of the cuff should be approximately 3/4"~1" (2~3 cm) above elbow. The tube should lie over the brachial artery on the inside of the arm. (Fig. 5)
4. Pull the cuff so that the top and bottom edges are tightened around your arm. (Fig. 6)
5. When the cuff is positioned properly, press the Velcro firmly against the pile side of the cuff.

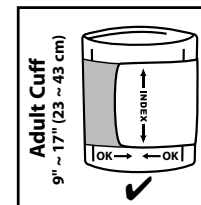
Fig. 7

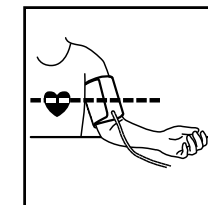
Fig. 8

6. This cuff is suitable for use if the <<INDEX>> mark falls within the <<OK RANGE>> marked by two arrows when the cuff is tightened around your arm. (Fig. 7)
7. Sit on a chair and place your arm on the table so that the cuff is at the same level as your heart. (Fig. 8)

Display BP4900/BP5900

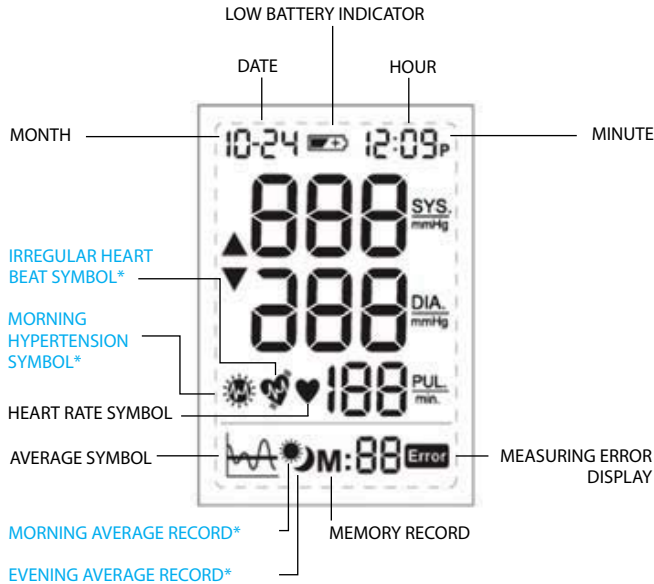
Fig. 9

* For BP5900 only

Note: On BP5900, the Backlight goes on when the unit is turned on, buttons are pressed, or when results are being displayed. It turns itself off after 10 seconds if no buttons are pressed.

Setting Month, Date and Time

- Press Mode (5) button ("month" starts flashing)
Press Set (4) button to set the correct month (1,2,3,...12)
- Press Mode (5) button ("date" start flashing)
Press Set (4) button to set the correct date (1,2,3,...31)
- Press Mode (5) button ("hour" starts flashing)
Press Set (4) button to set the correct hour (1,2,3,...12)
- Press Mode (5) button ("minute" starts flashing)
Press Set (4) button to set the correct minute (0,1,2,3,...59)
- Press Mode (5) button again to lock in time settings. Digits will stop flashing.

Taking a Measurement

Wrap the cuff around the arm (see "Applying the Arm Cuff" section above).

- Sit upright on the chair to have a correct posture.
- Press the "START/STOP" (1) button, all elements of the display indicate "888" during the first 3 seconds. The measuring blood pressure symbol will then flash on the display and the air pressure will automatically pump up to certain pressure level and start measurement. (Fig.9 BP4900/BP5900)
Do not move or talk in the midst of taking blood pressure measurement.
- After the air pressure has increased, it will slowly decrease. When the pulse is detected, the HEART RATE ♥ symbol will start flashing.

After taking blood pressure measurement, the product will turn off automatically after 1 minute.

Memory Function

Your blood pressure monitor can store the latest readings of 48 measurements for BP4900, or 99 measurements for BP5900.

Storing Measurement Data

After each blood pressure measurement, the Systolic pressure, Diastolic pressure, Pulse rate and the time & date of specific day will be automatically stored. Memory #01 is always the most recent one. Once the memory is full, the oldest values will be overwritten.

Recalling Stored Readings

- Press "Memory M" (2) button, M:00 will appear on the display if there is no data stored in the memory. If there is data stored in the memory, the latest recorded set of measurement which is the Systolic pressure, Diastolic pressure, Pulse rate, and the recording time and date and M:01 will appear on the display for 1 minute.
- Press the "Memory M" (2) button again to read the next line of stored data. The data called up on the display with 2-digit Memory number (ex. M:02).

7-Day Average Function

Full day Average

- Press the "Average" (3) button once, "Full Day Average" result will show on LCD.
- Make sure the DATE / TIME is correct when taking measurement, so you may get the correct 7-Day Average result.
- If there is no data in memory within past 7 days, "nA" will show on LCD.

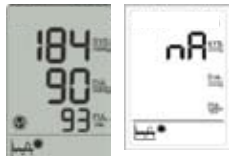
Morning Average (for BP5900 only)

- Press the "Average" (3) button again into "Morning Average" mode, the morning measurements in memory within past 7 days will show on LCD.
- If there is no data in memory within past 7 days, "nA" will show on LCD.


Evening Average (for BP5900 only)

- Press the "Average" (3) button once again into "Evening Average" mode, the evening measurements in memory within past 7 days will show on LCD.
- If there is no data in memory within past 7 days, "nA" will show on LCD.

Erasing Data

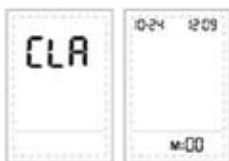- Press the "Memory" (2) button into Memory mode.
- Press and hold the "Mode" (5) button and then press the "Set" (4) button, all the stored data will be erased and CLR will show on screen.
- To check if the data was erased, press the "Memory" (2) button. No data should show up on screen, but date, time and M:00.

Morning Hypertension Setting (for BP5900 only)

Morning Hypertension symbol will appear when Morning Average is above your pre-configured Systolic rate and/or pre-configured Diastolic rate, AND/OR one or both of these measurements is 20% or more above Evening Average.

**** Your pre-configured Systolic / Diastolic should be decided between you and your doctor or healthcare provider.**

- After setting Date and Time, press "Mode" (5) button into "Morning Hypertension" setting mode "--" on Systolic position will flash for Systolic setting.
- Press "SET" (4) button, LCD will appear 135 mmHg for SYS as the internal pre-set value, then press "Set" (4) button repeatedly to set his own value (Range from 70~250 mmHg)
- Press "Mode" (5) button again into "Morning Hypertension" setting mode "--" on Diastolic position will flash for Diastolic setting.
- Press "SET" (4) button, LCD will appear 85 mmHg for DIA as the internal pre-set value, then user presses "SET" (4) button to set his own value (Range for DIA is 40 ~ 115 mmHg).
- User can press START/STOP (1) button to exit setting mode and go back to standby at any step.

Irregular Heart Beat Detector (for BP5900 only)

The appearance of this symbol signifies that a certain pulse irregularity was detected during the measurement. Talking, moving, shaking or an irregular pulse during the

measurement can result in the appearance of this icon. Usually this is not a cause for concern, however if the symbol appears often, we recommend you seek medical advice. The device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

As a safeguard, we recommend that if you have arrhythmias such as atrial or ventricular premature beats, atrial fibrillation or any other special conditions, you should check with your physician before using your blood pressure monitor.

Low Battery Indicator

When the low battery indicator flashes on the display, it means the battery is low and the four batteries need to be replaced with Alkaline LR6 (AA) batteries.

**** After replacing batteries, the Blood Pressure Monitor will go into TIME-SETTING mode automatically and show the last measurement time on screen. Please SET the current DATE / TIME before taking the next measurement to get correct AVERAGE result.**

Storage and Cleaning

- Always keep the unit in the carrying case after its use.
- Do not put the item directly under the sunlight, in high temperature or humid and dusty places.
- Do not store in extremely low (less than -4°F (-20°C)) or high (more than 140°F (60°C)) temperature.
- Use a piece of cloth with water or mild cleansing agent to clean the case and then use a piece of dry cloth to wipe it dry. Use a piece of dry cloth to wipe the cuff when it is dirty.
- Do not use any strong cleansers to clean it.
- When the unit is not to be used for a long time, remove the batteries. (Batteries may leak or cause harm).

What to do if ...

Problem	Reason	Solution
Heart Rate Symbol 	Appears in the measurement condition and flashes when pulse is detected.	• Measurement in progress, remain quiet.
Low Battery Indicator 	Appears when the battery voltage is excessively low or the positions of the batteries are incorrect.	• Replace all four batteries with new ones. Insert the batteries in the correct positions. Be aware of +/- positions.
Measuring Error 	Appears when the accurate blood pressure and pulse could not be obtained.	<ul style="list-style-type: none"> • Press "Start/Stop" button again and remeasure. • Check if cuff is wrapped according to instructions. • Check that there are no kinks in tube. • Check palm if exerting effort. • Check if talking or moving during measurement. • Check if posture is correct.

Specifications

Method of measurement	Oscillometric
Model number	BP4900, BP5900
Range of measurement	Pressure 0~300 mmHg Pulse 40~199 beats/minute
Accuracy	Pressure +/- 3 mmHg Pulse +/- 5% Max.
Inflation	Automatic
Exhaust	Automatic exhaust valve
Display	Liquid Crystal Display – Systolic, Diastolic, Pulse rate
Sets of memory	BP4900: 48 sets BP5900: 99 sets
Cuff size	Universal Cuff = 9"~17" (23~43 cm) arm circumference
Operating temperature	50°F (+10°C) ~ 104°F (+40°C), less than 85% R.H.
Storage temperature	-4°F (-20°C) ~ 158°F (+70°C), less than 85% R.H.
Unit weight	Approximately 2/3 lbs., 300g (without batteries)
Power supply	Alkaline Battery: 4 x AA (LR6)
Battery life	BP4900: 300 times measurement BP5900: 250 times measurement
Auto power off	Whenever not used for 1 minute
Accessories	4 batteries, arm cuff with tube, instruction manual, storage case

IMPORTANT

Read the operating instructions.

Classification:

- Internally powered equipment
- Type BF equipment
- IPX0
- Not suitable for use in presence of flammable anaesthetic mixture with air or with Oxygen or nitrous oxide
- Continuous operation with short-time loading

Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

Please do not dispose of the product in the household waste at the end of its useful life. Disposal can take place at your local retailer or at appropriate collection points provided in your country.

Warranty

Kaz grants 5 years warranty on the product commencing on the date of purchase. Within the warranty period we will eliminate, free of charge, any defects in the appliance resulting from faults in materials or workmanship, either by repairing or replacing the complete appliance as we may choose.

This warranty extends to every country where this appliance is supplied by Kaz or its appointed distributor.

This warranty does not cover: damage due to improper use, normal wear or use as well as defects that have a negligible effect on the value or operation of the appliance. The warranty becomes void if repairs are undertaken by unauthorized persons and if original Braun parts are not used. To obtain service within the warranty period:

Contact Us

Questions or comments?
Call toll-free: 1-800-327-7226
consumerrelations@kaz.com
www.kaz.com/braun

Consumer Relations
250 Turnpike Road
Southborough, MA 01772 USA

© 2009, Kaz, Incorporated - Made in China
Distributed by Kaz, Incorporated, Hudson, NY 12534 USA

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